ISOLANTE VIPI FILM

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Isolante

Indicação:

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Insulating Film

Indication:

Liquid for the insulation of gypsum in prosthetic work.

Characteristics / Benefits:

- For use in conventional and microwave techniques.
- Easy to apply due to its viscosity, it allows fluidity and fast drying, with excellent insulating properties.
- The film thickness is adequate, without causing dimensional alterations.
- Quick drying that optimizes working time.

Presentations:

Jars with 250 ml, 500 ml and 1000 ml.

Composition:

Alginate
Preservatives
Water

Instructions for Use:

Shake before using.

Use previously hydrated cast model.

During application, always use a clean brush covering the whole surface, being careful not to touch the posterior part of the teeth. When you make use of the separating film during the first pressing, remove it, trim the excess and apply ISOLANTE VIPI FILM. Carry out the insulation with the flask at 40°C.

Warnings / Precautions:

- Keep the bottle closed.
- Keep away from the light and heat.
- Keep out of the reach of children.
- Avoid contact with the skin and with the eyes.
- Do not contaminate the jar with cast residues.
- Inflammable.
- If there is irritation, seek medical attention.
- If there is contact with the eyes, wash with plenty of water.
- If ingested accidentally, drink water and do not provoke vomiting. Seek medical attention.

Shelf Life:

2 years as of the date of manufacture.

Product declared exempt from registration by the Ministry of Health.

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