Removedor de ceras

Indicação:

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Wax remover

Indication:

Elimination of wax residues and oiliness in the elaboration of dentures and cleaning of oiliness : greases in general.

Characteristics / Benefits:

- For use in conventional and microwave techniques.
- It eliminates residues of waxes, oiliness and greases in general, contributing significantly to adherence of the teeth to the denture's base.
- Totally odorless, without leaving residues on the hands, instruments or denture.
- Only product in the market.

Presentation:

500 ml bottle

Composition:

Saturated Aliphatic Hydrocarbon

Instructions for Use:

After inclusion in the Muffle and curing of the plaster, the wax must be removed, once the teeth are already positioned.

For this purpose, heat the water in a separate vessel.

Immerse the Muffle for about 5 minutes in this vessel and open to eliminate the melted wax.

Wash the internal surfaces of the Muffle with a cotton ball soaked in REMOX, rubbing through the whole area, to remove wax residues. This cleaning operation must be repeated, if necessary.

In case you make use of MUFLA VIPI STG for microwaves, perform the heating in the microwave oven for 3 minutes at low power. To remove the wax residues, moisten the cotton with REMOX, rubbing on the area to be cleaned. For even more complete cleaning, fill the Muffle cavity with a dry ball of cotton; close the Muffle and then, repeat the heating operation in the microwave oven.

Warnings / Precautions:

- Flammable and volatile product.
- Keep the bottle closed.
- Use far from fire.
- Do not apply on warm surfaces.
- Keep out of the reach of children.
- Avoid contact with the skin and with the eyes.
- If it causes irritation, wash with running water.
- If ingested accidentally, drink water and do not provoke vomiting. Seek medical attention.
- Handling of the product in ventilated places is recommended, by using nitrile gloves and protective goggles.
- Their handling is not recommended to pregnant women, nursing mothers and people with known allergies to this type of product.
- Conserve in dry and fresh place.

Shelf Life:

5 years as of the date of manufacture.

Product declared exempt from registration by the Ministry of Health.

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