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C.N.P.J.(MF) 49.425.259/0001-73

0800-7712226
www.vipi.com.br

STG GENGIVAS (Gum Characterization)

Indication:

Prosthesis characterization according to the patient's gingival color.

Contraindications:

Material contains trace amounts of methyl methacrylate monomer which may cause skin sensitization (allergic contact dermatitis) or other allergic reactions in susceptible persons.

Warnings / Precautions:

- For dental use only.
- Liquid: Flammable, non-toxic material (Flash point: 10°C/50°F).
- Resin: May form flammable air/dust mixtures.
- As with all acrylic resins, it is recommended to handle the product in ventilated areas, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies to this type of product.
- May cause skin irritation in susceptible people.
- VIPI recommends for any acrylic resin only mechanical polishing.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.
- The use of wax thinner is not recommended. Residual solvent can cause less bonding of teeth to acrylic base.
- Follow the usual procedures for microbiological control, assembly, articulation and maintenance of the prosthesis, as well as for individual protection.
- Do not let non-polymerized material come into contact with the patient's mucosa. The heat of polymerization can cause burns.
- Do not dispose of waste into sewers and water courses or in domestic waste.
- Dispose of the product according to local regulations.
- PROHIBITED TO REPROCESS

Storage Conditions:

Store protected from light and heat, closed in a dry and cool place.

Instructions for Use:

Prior to starting any work, the containers must be completely clean and free of traces of wax or grease. See technical book in Annex.

Characteristics / Benefits:

- It constitutes a working system for containing written method that allows repetition of characterizations scientifically.
- 16 predefined patterns.
- The method allows the customization of denture in accordance with the patient's gum coloration.
- For use in conventional and microwave techniques.
- Absence of porosities.
- Low water absorption.
- Low residual monomer.
- Excellent mechanical properties.
- Excellent dimensional stability.
- Unmatchable brilliance.
- Ease of drainage, compaction, ease in finishing, polishing and brilliance.
- It can be used with VIPI CRIL PLUS (with Crosslink additive) or VIPI WAVE fluids.

Presentations:

Powder: Bottle 80g

KIT WITH:

6 bottles of powdered resin of 15g each;

2 vein bottles;

Technical Manual with method of application

Observation: The resins and veins are also sold separately for replacement.

Colors:

POWDER: Red Pink, Black Pink, Light Pink, Brown Pink, Purple Pink and Medium Pink.

POWDER (ATYPICAL COLORS): Purple 1 R1FS, Red 1 V1FS, Red 2 V2FS

VEINS: Red and Purple.

Composition:

POWDER

VEINS

Polymethylmethacrylate

Rayon Thread

Pigments

Pigments

Recommendations to patients:

To increase the durability of the prosthesis, daily hygiene is recommended without the use of aggressive substances, such as acids or alkalis.

Shelf Life: 10 years as of the date of manufacture.

ANVISA N°: 10216040028

Technical Responsible: Desiree M. Cortez - CRF-SP-41566

Additional information:

- Check the correlation of the version of the instruction for use with the purchased product indicated on the packaging.
- To obtain the Instruction for Use in printed form at no additional cost, ask the customer service (SAC) by calling 0800 771 2226 (Brazil only) or sending an email to atendimento@dentsplysirona.com

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