Resina Acrílica

Indicação:

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Resina Acrílica

Cores:

1A, 1D, 2A, 2B, 3E, 4B, INCISAL I, INCISAL II, CERVICAL I, CERVICAL II E CERVICAL III.
A1, A2, A3, A3,5, A4, B2, INCISAL V1, INCISAL V2, INCISAL V3, CERVICAL V1, CERVICAL V2, CERVICAL V3.

KIT (1A,2A,3E)
Dentinhas 1A, 2A, 3E, Incisal I,
Cervical I e Cervical III (20g)
Líquido fotopolimerizável (15 ml)

KIT (1D,2B,4B)
Dentinhas 1D, 2B, 4B, Incisal II,
Cervical I e Cervical II (20g)
Líquido fotopolimerizável (15 ml)

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Acrylic Resin

Indication:

Bivalent polymerizable acrylic resin for making crowns, bridges, fixing vests and jackets, PTR and PPR retention fills and cavities in the work of characterizing acrylic teeth.

Contraindications:

There are no known adverse side effects when the product is used according to current procedures.

Warnings / Precautions:

For dental use only.

- Liquid: Flammable, non-toxic material (Flash point: 10°C / 50°F).
- Resin: Can form flammable air / dust mixtures.
- As in all acrylic resins, it is recommended to handle the product in ventilated places, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies for this type of product.
- Keep away from light and heat.
- Do not dispose of waste in sewers and waterways or household waste.
- VIPI recommends only mechanical polishing for any acrylic resin.
- Products that are not compatible with the resin / monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.

Instructions for Use:

PREPARATION:

Prior to starting any work, the containers must be completely clean and free of traces of wax or grease.

The vessels and the hands must be contaminant free to avoid color alterations. Shake well the bottle before using.

DOSING AND PLASTICIZATION:

The recommended powder / fluid proportion is 2 measures of powder to 1 measure of fluid.

To obtain colors and results closest to the scale, it is important to observe the right proportion between the body and the incision, knowing that to whiten the crown's hue, the overlay of the incision (enamel) is extended along the gingival third. To darken, limit the layer to the medial third, exposing more the body's color.

The other option is the Powder and Fluid system when a layer of resin is placed and then, with dropper, it is saturated with monomer and immediately a new layer of resin is applied, until the desired thickness is reached in the work. In this method, a total saturation with monomer over the resin is essential so that it may not dry up and in the previous application, take the work to the polymerizer still in the plastic phase.

POLYMERIZATION:

Works done with Selfpolymerizable Fluids must be taken to the hydropneumatic polymerizer to reach complete polymerization. In a normal cycle, polymerize for 20 minutes at a pressure of 20 pounds.

For works with photopolymerizable fluids, prepare the resin in a deppan jar and apply with a semiflexible spatula over the area to be repaired. Place in a photopolymerizer for 2 minutes. The resin layers must be polymerized at a thickness of 1.5 mm.

FINISHING AND POLISHING:

Start finishing with bits and mills, pass through abrasive stones and conclude with fine textured sandpapers, perform polishing with soft brush and VIPI BRIL ACABAMENTO and POLIMENTO

Characteristics / Benefits:

- Resin with Trilux quality the best 3 layers of all times.
- Rigorous selection and raw material receipt tests combined with successive tests during all manufacturing cycles, under the demanding ISO 13485 standards, assure the specified results in the final product.
- Greater mechanical, chemical and abrasion resistance.
- The self and photopolymerizable fluids allow flexibility and speed in works.
- It supports the most varied polymerization methods.
- Broad range of colors, assuring repeatability and customization of your works.
- The fluids with Fluorescent and Crosslink provide durability and natural aesthetics.
- Uniform polymerization in all parts of the denture, even in the thicker parts.
- Ease in application for execution of work, finishing, polishing and brilliance.
- It complies with the specifications of ISO 20795-1

Presentations:

POWDER: 20g.

SELF-POLYMERIZABLE FLUID: 50ml.

PHOTOPOLYMERIZABLE FLUID: 15ml.

TRILUX COLOR

Technical Datasheet
English

Acrylic Resin

Colors:

1A, 1D, 2A, 2B, 3E, 4B, INCISOR I, INCISOR II, CERVICAL I, CERVICAL II AND CERVICAL III
A1, A2, A3, A3,5, A4, B2, INCISOR V1, INCISOR V2, INCISOR V3, CERVICAL V1, CERVICAL V2 AND CERVICAL V3

KIT (1A, 2A, 3E)

Dentins 1A, 2A, 3E, Incisor I,
Cervical I and Cervical III (20g)
Photopolymerizable fluid (15ml)

KIT (1D, 2B, 4B)

Dentins 1D, 2B, 4B, Incisor II,
Cervical I and Cervical II (20g),
Photopolymerizable fluid (15 ml)

KIT (A1, A2, B2)

Dentins A1, A2, B2, Incisor V1,
Cervical V1 and Cervical V2 (20g)
Photopolymerizable fluid (15ml)

KIT (A3, A3,5, A4)

Dentins A3, A3,5, A4, Incisor V2,
Incisor V3 and Cervical V3 (20g),
Photopolymerizable fluid (15 ml)

COLORS	DENTIN	INCISOR	CERVICAL
1A	DENTIN 1A	INCISOR I	CERVICAL I
1D	DENTIN 1D	INCISOR II	CERVICAL I
2A	DENTIN 2A	INCISOR I	CERVICAL I
2B	DENTIN 2B	INCISOR II	CERVICAL II
3E	DENTIN 3E	INCISOR I	CERVICAL III
4B	DENTIN 4B	INCISOR II	CERVICAL II

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Composition:

POWDER
Polymethylmethacrylate
Benzoyl Peroxide
Pigments

SELF-POLYMERIZABLE FLUID
Methylmethacrylate
EDMA (Crosslink)
Inhibitor
Fluorescent Pigments
DMT

PHOTOPOLYMERIZABLE FLUID
Methylmethacrylate
EDMA (Crosslink)
Inhibitor
Fluorescent Pigments
DMT

Shelf Life:

RESIN: 10 years as of the date of manufacture.
FLUIDS: 2 years as of the date of manufacture.

ANVISA Registration Nº:

RESIN: 10216040031
PHOTOPOLYMERIZABLE / SELF-POLYMERIZABLE FLUID: 10216040032

Technical Responsible: Luiz Paulo Ribeiro Junior - CRQ IV (Regional Council of Chemistry) 03251745

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TRILUX COLOR

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Couleurs:

1A, 1D, 2A, 2B, 3E, 4B, INCISAL I, INCISAL II, CERVICAL I, CERVICAL II ET CERVICAL III.
A1, A2, A3, A3,5, A4, B2, INCISAL V1, INCISAL V2, INCISAL V3, CERVICAL V1, CERVICAL V2, CERVICAL V3.

KIT (1A,2A,3E)
Dentines 1A, 2A, 3E, Incisal I,
Cervical I et Cervical III (20g)
Liquide photopolymérisable (15 ml)

KIT (1D,2B,4B)
Dentines 1D, 2B, 4B, Incisal II,
Cervical I et Cervical II (20g)
Liquide photopolymérisable (15 ml)

KIT (A1, A2, B2)
Dentines A1, A2, B2, Incisal V1,
Cervical V1 et Cervical V2 (20g)
Liquide photopolymérisable (15 ml)

KIT (A3, A3,5, A4)
Dentines A3, A3,5, A4, Incisal V2,
Incisal V3 et Cervical V3 (20g)
Liquide photopolymérisable (15 ml)

COULEURS	DENTINE	INCISAL	CERVICAL
1A	DENTINE 1A	INCISAL I	CERVICAL I
1D	DENTINE 1D	INCISAL II	CERVICAL I
2A	DENTINE 2A	INCISAL I	CERVICAL I
2B	DENTINE 2B	INCISAL II	CERVICAL II
3E	DENTINE 3E	INCISAL I	CERVICAL III
4B	DENTINE 4B	INCISAL II	CERVICAL II

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