PORTUGUÊS

Indicação:

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ENGLISH

Indication:

WAX block for milling casting or injection patterns (substructures and crowns cemented or screwed), created and milled by CAD/CAM system.

Composition:

Polyethylene wax and colophony.

Validity:

10 years from the manufacturing date.

Presentation:

1 piece

Heights (according to the table annexed to the models):

Colors:

Material available in gray and ivory colors.

Use instructions:

The VIPI BLOCK WAX should be fixed in the milling machine according to the instructions from the device manufacturers.

Important Note:

Avoid overheating during machining and finishing.

Recommended operating temperature: Between 20°C (68°F) to 30°C (86°F).

Limits:

- Posteriors: Do consider indications of material to be done, as metal (casting), ceramics pressing, etc.
- Minimum thickness: 1 mm occlusal and cervical 0.6mm
- Minimum Diameter connection: Anteriors: 3.6mm and Posterior: 4.4 mm

Benefits:

- Special high quality milling wax;
- 100% burnout;
- Reduced milling time;
- Easy extraction;
- Ideal for casting and injection processing;
- Excellent machinability;
- Low tool consumption during milling process;
- Good contrast.

Storage:

- Keep away from heat and light.
- The product colors may vary due to the use of natural wax.

Product declared exempt from registration by the Ministry of Health.

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