VIPI CRIL PLUS

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Resina Acrílica

Indicação:

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Acrylic Resin

Indication:

Thermopolymerizable Acrylic Resin for Denture Bases. It is applied to Removable Total Denture, Permanent Total Overdenture, Removable Partial Denture and Rebases, To replace part of chewing system of the human body.

Contraindications:

Material contains trace amounts of methyl methacrylate monomer which may cause skin sensitization (allergic contact dermatitis) or other allergic reactions in susceptible persons.

Warnings/Precautions:

- For dental use only.
- Liquid: Flammable, non-toxic material (Flash point: 10°C/50°F).
- Resin: May form flammable air/dust mixtures.
- As with all acrylic resins, it is recommended to handle the product in ventilated areas, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies to this type of product.
- May cause skin irritation in susceptible people.
- VIPI recommends for any acrylic resin only mechanical polishing.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.
- The use of wax thinner is not recommended. Residual solvent can cause less bonding of teeth to acrylic base.
- Follow the usual procedures for microbiological control, assembly, articulation and maintenance of the prosthesis, as well as for individual protection.
- Do not let non-polymerized material come into contact with the patient's mucosa. The heat of polymerization can cause burns.
- Do not dispose of waste into sewers and water courses or in domestic waste.
- Dispose of the product according to local regulations.
- PROHIBITED TO REPROCESS.

Storage Conditions:

Store protected from light and heat, closed in a dry and cool place.

Step-by-Step Instructions:

Prior to starting any work, the containers must be completely clean and free of traces of wax or grease.

The mixing vessels, hands and spatula must be contaminant-free to avoid color alterations. We recommend a room temperature for mixing and application between 15°C and 30°C.

Prepare the VIPI CRIL PLUS resin by observing the correct proportion between powder and liquid: place in a mixing vessel, preferably glass, 6.5 ml of VIPI CRIL PLUS monomer (smaller measurement of the VIPI measuring utensil), preferably with Crosslink. Then, add 14 g (19 ml) of resin in the desired color (larger measurement of the VIPI measuring utensil). With a plastic spatula, mix until the mixture is well homogeneous. Wait for the plastic phase, which will take place when the material loosens from the vessel walls (10 min after dosing at 23°C) to start the inclusion.

PRESSING:

After reaching the plastic phase, the resin must be applied in the previously insulated muffle in the preferred manner and, covered by a high density polyethylene plastic film, never cellophane. Take to the press MUFLA VIPI STG to perform the pressing test (slowly up to a pressure of 1000 kgf). After removing the plastic film and the excess material, the muffle must be closed again and definitive pressing must be performed (1000 kgf). Before polymerizing, the resin must rest for 20 minutes. To increase the translucency of the resin and to avoid porosities, it is recommended to allow the resin to rest for 2 hours, before starting the polymerization, in case you are using metal muffle, follow its manufacturer's recommendations.

POLYMERIZATION CYCLES:

After the definitive pressing, place the common muffle in the cooking press (clamp) or, in the case of the MUFLA VIPI STG, close with 4 screws and bring to polymerization. The VIPI CRIL PLUS resin is very versatile, which can be used in various polymerization cycles. Two (2) polymerization examples are presented next to obtain a residual monomer content less than 1%:

CONVENTIONAL POLYMERIZATION:

Place the muffle in a pot with 3 liters of cold water, turn on the stove with low flame, allowing it to reach the temperature of 70°C. Maintain this temperature for 30 min. Increase the flame to reach 100°C, maintain the boiling for one hour and a half and, then put out the flame. Allow the muffle to cool down inside the water (up to 40°C) for approximately 20 minutes, before starting the deflasking.

THERMO-PNEUMATIC POLYMERIZATION:

Place the muffle in the polymerizer, covering it with water. Close and inject compressed air (60 pounds). Turn on the electric power, until it reaches 120°C. As soon as this temperature is reached, turn off and allow it to cool down up to 90°C. Once this temperature is reached, turn on again until it reaches 120°C. Turn off and allow it to cool down up to 60°C. Make sure that there is no pressure inside the polymerizer, open and withdraw the muffle, to start the deflasking.

DESFLASKING:

Deflasking is executed normally, making sure that the muffle is cold and, carefully so as not to impair the teeth or the denture.

FINISHING AND POLISHING:

Finishing is started with bits and mills, passing to abrasive stones and concluded with fine textured sandpapers, and concluded with finishing rods and brush at least no. 10, 12 or 27. Polishing is performed with flannel brush and polishing rods.

NOTE: The thermopolymerizable acrylic liquid VIPI CRIL PLUS is found in two options, normal or with Crosslink additive, which endows to the product greater impermeability, impeding impregnations, unpleasant odors and, contributing toward obtainment of unalterable dentures in the buccal medium and non irritating.

The use of VIPI CRIL PLUS with Crosslink makes polishing and brilliance easier, providing its greater durability. VIPI recommends for this product, solely mechanical polishing.

Acrylic Resin

Characteristics / Benefits:

- Rigorous selection and raw material receipt tests combined with successive tests during all manufacturing cycles, under ISO standard requirements, assure the specified results in the final product.
- It supports the most varied professionally known polymerization methods for thermopolymerizable resins.
- The high molecular weight makes handling and application easier, assuring excellent results in your work.
- The range of colors is the most complete in the market, attending to all variables.
- The color scale in gum form, makes the choice of adequate color easier.
- Selection and control of pigments assure high color stability.
- Appropriate packaging for use of the supplied measuring utensils, which makes measurement of components easier, separately, and the conservation of the product until the end.
- VIPICRIL PLUS resin is acrylate free, reducing the possibility of allergic reactions.
- Liquid with Crosslink assuring greater product resistance.
- Ease in handling and application for execution of work.
- Uniform polymerization in all parts of the denture, even in the thick parts.
- Low water absorption, low residual monomer, excellent mechanical properties, high impact resistance, unmatched brilliance, absence of porosities, ease of drainage, compaction and drainage.

Product Configurations:

POWDER: Box with 50 doses of 14 g each; Bottles of 80g, 225g, 450g, 1Kg, 2,250kg, 5kg; Barrel of 25kg and 50kg.

LIQUID: (With or Without Crosslink): Bottles of 50ml, 120ml, 250ml, 500ml and 1000ml; Can of 900mL and 5L

Shades:

WITH VEINS: Light Pink, Medium Pink, Medium Pink-B, Dark Pink, Special Pink and Black.

WITHOUT VEINS: Colorless, Crystal and Palate.

Composition:

POWDER	LIQUID
Polymethylmethacrylate	Methylmethacrylate
Polypropylene	EDMA (Crosslink)
Pigments	Inhibitor

Infection control procedure

INSTRUCTIONS FOR DENTURE DISINFECTION

Note: Instructions are not intended for routine denture care.

	Method	Warning
Cleaning & Disinfection: Manual	<ul style="list-style-type: none"> • Saturate a lint-free wipe with an alcohol quaternary-based solution. • Wipe the entire appliance ensuring to wipe all grooves and edges well. • Spray the entire surface with the alcohol quaternary-based solution to thoroughly wet the appliance - <i>maintain wet for the contact time as recommended per the manufacturer's instructions</i>. • After the manufacturer recommended contact time has elapsed, rinse the appliance under potable water for a minimum of 30 seconds. • Allow the appliance to air dry completely. 	*Follow the manufacturer's instructions for solution contact time.

Recommendations to patients:

To increase the durability of the prosthesis, daily hygiene is recommended without the use of aggressive substances, such as acids or alkalis.

Shelf Life:

POWDER: 10 years as of the date of manufacture. | LIQUID: 2 years as of the date of manufacture.

ANVISA N.º:

POWDER: 10216040028 | LIQUID: 10216040033

Technical Responsible: Desiree M. Cortez - CRF-SP-41566

Additional information:

- Check the correlation of the version of the instruction for use with the purchased product indicated on the packaging.
- To obtain the Instruction for Use in printed form at no additional cost, ask the customer service (SAC) by calling 0800 771 2226 (Brazil only) or sending an email to atendimento@dentsplysirona.com

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