Document Detail

Type: 1100-PIR-DFU
Document No.: 1100-PIR-DFU 0014[5]
Title: FICHA TÉCNICA VIPIWAVE
Comment:
Status: CURRENT
Effective Date: 24-Jul-2023

Approval

<u>Owner Role</u>	<u>Sign-off By</u>	<u>Sign-off Date</u>	
1100-PIR-REGULATÓRIO Aprovador Brasil Pirassununga Regulatório Aprovador	Ana Carolina Nilson	24-Jul-2023 9:18 pm	GMT
1100-Marketing Approver Marketing Approver	Fabio Domingues	24-Jul-2023 7:17 pm	GMT
1100-Document Control Document Control	Priscilla Silva	24-Jul-2023 6:50 pm	GMT
1100-PIR-ENGENHARIA Aprovador Brasil Pirassununga Engenharia Aprovador	Bruno Conti	24-Jul-2023 6:56 pm	GMT
1100-PIR-QAAR Aprovador Brasil Pirassununga QAAR Aprovador	Gabriel Ocanha	24-Jul-2023 9:08 pm	GMT
1100-Research and Development Brazil Research and Development	Luiz Crepaldi	24-Jul-2023 7:07 pm	GMT
1100-Clinical Education Appro Clinical Education Approver	Ana Zimbaldi	24-Jul-2023 6:49 pm	GMT

Resina Acrílica

Indicação:

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Patient recommendations:

To increase the longevity of the prosthesis, it is recommended to maintain daily hygiene without the use of aggressive substances such as acids or alkalis.

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Acrylic Resin

Indication:

Thermopolymerizable Acrylic Resin AND Fluid for Denture Bases. It is applied to Removable Total Denture, Permanent Total Overdenture, Removable Partial Denture (RPD) and Rebases.

Contraindications:

Material contains trace amounts of methyl methacrylate monomer which may cause skin sensitization (allergic contact dermatitis) or other allergic reactions in susceptible persons.

Warnings/Precautions:

- For dental use only.
- Liquid: Flammable, non-toxic material (Flash point: 10°C/50°F).
- Resin: May form flammable air/dust mixtures.
- As with all acrylic resins, it is recommended to handle the product in ventilated areas, using nitrile gloves and protective goggles; however, handling is not recommended for pregnant women, nursing mothers and people with recognized allergies to this type of product.
- May cause skin irritation in susceptible people.
- VIPI recommends for any acrylic resin only mechanical polishing.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, petroleum jelly and other solvents.
- The use of wax thinner is not recommended. Residual solvent can cause less bonding of teeth to acrylic base.
- Follow the usual procedures for microbiological control, assembly, articulation and maintenance of the prosthesis, as well as for individual protection.
- Do not let non-polymerized material come into contact with the patient's mucosa. The heat of polymerization can cause burns.
- Do not dispose of waste into sewers and water courses or in domestic waste.
- Dispose of the product according to local regulations.
- PROHIBITED TO REPROCESS

Storage Conditions:

Store protected from light and heat, closed in a dry and cool place.

Step-by-Step Instructions:

Shake the bottle before using.

PREPARATION:

The Silicon for wall and the Insulating assure dentures free from plaster incrustations, make deflasking and complete polymerization of acrylic resin easier.

The vessels and the hands must be contaminant free to avoid color alterations. The wax model is prepared and included in the MUFLA VIPI STG normally.

The wax is melted and the Muffle cavity washed with hot water and detergent or in the manner chosen by the professional. The cavity and the plaster base are insulated with an Alginate based insulating agent.

DOSING AND PLASTICIZATION:

The exclusive use of the VIPI WAVE Fluid is recommended to maintain the properties and the specified quality for this product.

Place 6.5 ml of the monomer (smallest measurement of the VIPI measuring utensil) in the mixing vessel, preferably glass. Add 14 g (19 ml) of the resin in the desired color (the largest measurement of the VIPI measuring utensil) and with a plastic spatula to mix the resin and the monomer until perfect homogenization. Wait for the plastic phase which takes place when the material is set loose from the vessel walls.

Alternative - Place the necessary amount of monomer in the vessel and saturate with the resin in the desired color.

Note - It is important to conserve the moisture content of the Muffle. In cases of works that will be carried out within a day, keep it hydrated.

PRESSING:

In the plastic phase, the resin must be included in the Muffle insulated in the preferred manner. The resin is covered by a separating plastic film (high density polyethylene, never cellophane!). Take to the Muffle to the press and perform the pressing test (slowly up to a pressure of 500 / 1000 kgf). After removing the separating plastic and the excess material, close the Muffle and perform the definitive pressing (at most 1000 Kgf). For dentures of at most 10 mm or for dentures done with the use of Silicon wall, wait for 15 minutes in the press or outside of it, before starting the polymerization in the microwave. In the case of dentures with thickness above 10 mm, a waiting time of 2 hours is recommended. For dentures above 20 mm the waiting time of 2 hours is indispensable.

Alternative - Instead of waiting for the plastic phase, some professionals pour the well liquid resin in the Muffle, cover with separating plastic and wait for the arrival of the plastic phase inside the Muffle before starting the pressing.

POLYMERIZATION:

Polymerization must be done rigorously, within the parameters defined in the table below:

	For 500W oven		For 800 to 1100W oven		For 1200 to 1400W oven	
INITIAL STAGE	20 min	With 20/30% Power	20 min	With 10/20% Power	20 min	With 10% Power
FINAL STAGE	+ 5 min	With 80/100% Power	+ 5 min	With 50/60% Power	+ 5 min	With 30/40% Power

Observations - Polymerization cycles of the STG System can also be used, according to instructions in the product manual.

DEFLASKING:

Deflasking is executed normally, making sure that the muffle is cold and, carefully so as not to impair the teeth or the denture. For greater ease in deflasking the use of Silicon for the wall and Insulating is recommended

Acrylic Resin

FINISHING AND POLISHING:

Finishing is started with bits and mills, passes to abrasive stones and is concluded with fine textured sandpapers and finishing rods. Polishing is performed with soft brush and polishing rods..

Characteristics/Benefits:

- Rigorous selection and raw material receipt tests combined with successive tests during all manufacturing cycles, under ISO standard requirements, assure the specified results in the final product.
- It supports the most varied professionally known polymerization methods for thermopolymerizable resins.
- The high molecular weight makes handling and application easier, assuring excellent results in your work.
- The range of colors is the most complete in the market, attending to all variables.
- The color scale in gum form, makes the choice of adequate color easier.
- Selection and control of pigments assure high color stability.
- Appropriate packaging for use of the supplied measuring utensils, which makes measurement of components easier, separately, and the conservation of the product until the end.
- VIPICRIL PLUS resin is acrylate free, reducing the possibility of allergic reactions.
- LIQUID with Crosslink assuring greater product resistance.
- Ease in handling and application for execution of work.
- Uniform polymerization in all parts of the denture, even in the thick parts.
- Low water absorption, low residual monomer, excellent mechanical properties, high impact resistance, unmatched brilliance, absence of porosities, ease of drainage, compaction and drainage.

Product Configurations:

POWDER: Bottles of 225 g and 450g.

LIQUID: Bottles of 120ml and 250ml.

Shades:

WITH VEINS: Medium Pink.

WITHOUT VEINS: STG Palate and Colorless.

Composition:

POWDER	LIQUID
Polymethylmethacrylate	Methylmethacrylate
Polypropylene	EDMA (Crosslink)
Pigments	Inhibitor
	Terpinolene

Infection control procedure

INSTRUCTIONS FOR DENTURE DISINFECTION

Note: *Instructions are not intended for routine denture care.*

	Method	Warning
Cleaning & Disinfection: Manual	<ul style="list-style-type: none"> • Saturate a lint-free wipe with an alcohol quaternary-based solution. • Wipe the entire appliance ensuring to wipe all grooves and edges well. • Spray the entire surface with the alcohol quaternary-based solution to thoroughly wet the appliance - <i>maintain wet for the contact time as recommended per the manufacturer's instructions.</i> • After the manufacturer recommended contact time has elapsed, rinse the appliance under potable water for a minimum of 30 seconds. • Allow the appliance to air dry completely. 	<p>*Follow the manufacturer's instructions for solution contact time.</p>

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Shelf Life:

POWDER: 10 years as of the date of manufacture. | LIQUID: 2 years as of the date of manufacture.

ANVISA nº.:

POWDER: 10216040028 | LIQUID: 10216040033

Technical Responsible: Desiree M. Cortez - CRF-SP-41566

Additional information:

- Check the correlation of the version of the instruction for use with the purchased product indicated on the packaging.
- To obtain the Instruction for Use in printed form at no additional cost, ask the customer service (SAC) by calling 0800 771 2226 (Brazil only) or sending an email to atendimento@dentsplysirona.com

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